AccuTEC®

AccuTec, Inc.

One Razor Blade Lane Verona, VA 24482-0979 USA
info@accuthrive.com www.accuthrive.com

CE 2797

CH REP

Casus Switzerland GmbH
Hinterbergstrasse 49
6312 Steinhausen
Switzerland

EC REP

Emergo Europe
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands

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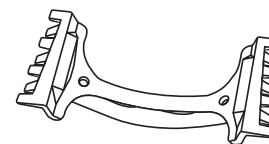
DermaBlade®

En

Instructions for Use, Storage
and Handling Recommendations

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AccuTEC®

DermaBlade® Storage and Handling Recommendations

En

For best results, AccuThrive recommends the following practices for safe and effective storage and handling of DermaBlade and other medical devices.

- DermaBlade should remain in sealed pouch until needed for use. Do not use if package is damaged.
- Store all cleaning and disinfecting chemicals containing chlorides, fluorides or other strong oxidizing agents in a separate enclosed area.
- Environmental controls should be set to maintain relative humidity between 30 and 60%.
- Do not store liquids above dry medical devices.
- Do not store this medical device on the ground, near a window, under the sink or in proximity to other sources of humidity.

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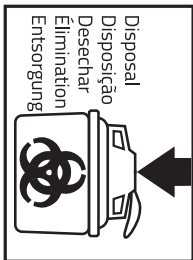
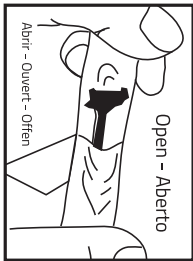
DermaBlade®

Instructions for Intended Use: Shave Biopsy

Read Instructions
Before use.

En

1. Cleanse/disinfect area in the usual manner.
2. Achieve local anesthesia.
3. Remove DermaBlade® from sterile package. Do not use if package is damaged.
4. Grip between thumb and first two fingers of gloved, dominant hand.
5. Flex blade to angle necessary to perform procedure.
6. Excise lesions with a side-to-side slicing motion of the blade, advancing with gentle forward pressure.
7. Dispose of used blade in sharps container.
8. Warning: Blade is extremely sharp. Handle with care. Used sharps are contaminated. Disease may occur from contact or injury. Dispose in puncture-resistant container. Refer to local guidelines for handling and disposal of sharps.

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©2026 AccuTec®, Inc.